USER MANUAL

PLIKO

MANUALE D'USO

fumagalli
WE TAKE CARE

MANUALE D'USO

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USER MANUAL

WARRANTY

Please refer to the enclosed certificate.

MAINTENANCE

- Removable and washable cover up to 30°
- Keep the movement parts clean and greased using a light oil.
- Dry the metal parts in order to avoid rust.
- Periodically clean the plastic parts with a moist cloth: do not use solvents or other similar products
- Brush the dust off from textile parts.
- Keep the wheels clean from dust and sand.
- Protect the product from adverse weather conditions (rain or snow)- continuous exposure to sunlight could cause some of the materials to change colour.

Warning! Do not use chlorine-based detergents.

SAFETY


- Never leave your child unattended.
- Avoid putting fingers into the blocking parts.
- Unfolding of the buggy should only be carried out by an adult.
- Do not use the buggy if parts are missing or broken.
- Always use the brake when the buggy is still.
- Do not use the buggy near stairs or steps.
- Do not lift the buggy by its front bar as it is not designed to withstand this. If you need to lift the buggy, lift it from under the armrests or by the frame
- Ensure that the buggy is correctly opened and blocked: the red signs under the block levers should not be visible.

**CE conforming to European directives
on medical devices
93/42/CEE – CLASS 1 -**

Fumagalli S.r.l. takes no responsibility for any damage arising from the failure to comply with the instructions of the user manual. For any further questions contact our technical department at Fumagalli srl Tel. +39 031 3356811.

Furthermore, with the intent of improving its products, Fumagalli S.r.l reserves the right to bring about modifications to the functional and aesthetic aspects of its products.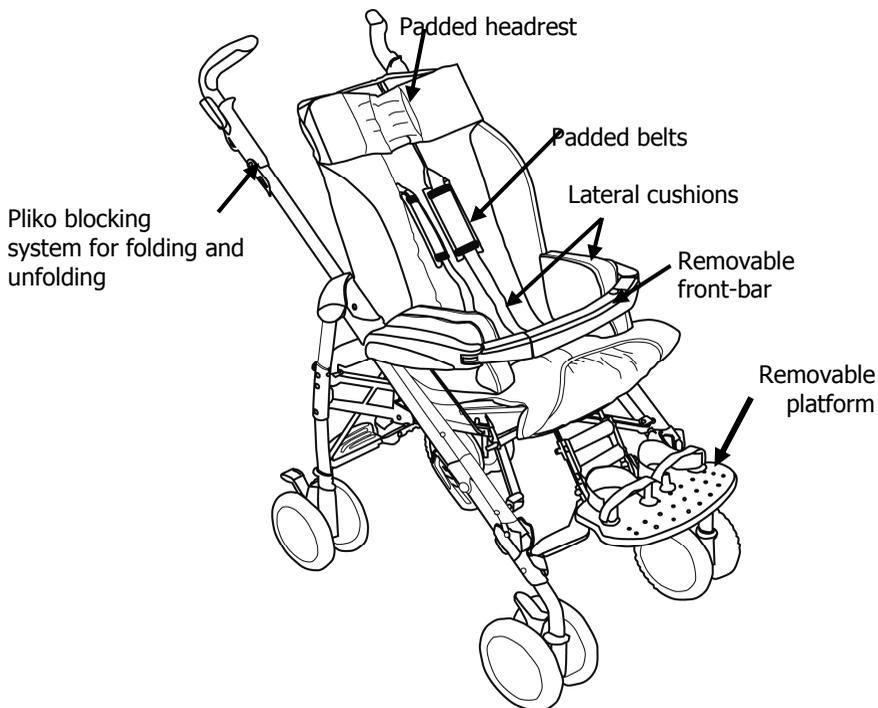


WARNING!
ALWAYS REMOVE THE FOOTREST BEFORE FOLDING

Non-removal of the footrest before folding may cause serious, irreparable structural damage. Fumagalli therefore accepts no responsibility for any damage arising from failure to comply with the instructions written in the user manual.

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
Pull up the two handle levers and at the same time lift the buggy which will then unfold itself automatically. Check the two side hooks to verify that the buggy is completely open.

After unfolding, position the footrest platform and the lateral padded cushions if they have unhooked during transportation.

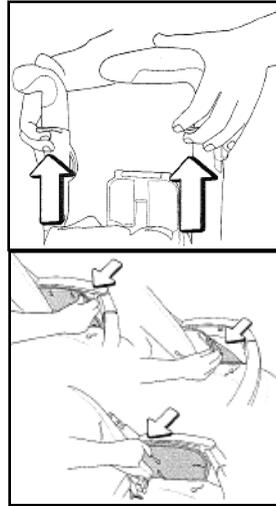
Side hooks

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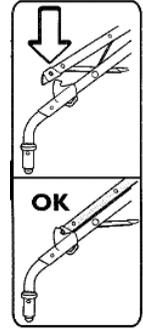To fold the buggy, remove the footrest platform by pulling its blocking cable downwards; then pull up the two levers on the push-handles so that the hooks are also released. Hold the back-handle (3) and pull it up: the buggy will fold itself automatically. Now the buggy can be pulled along on its wheels or carried by its handle.


Blocking cable of the footrest platform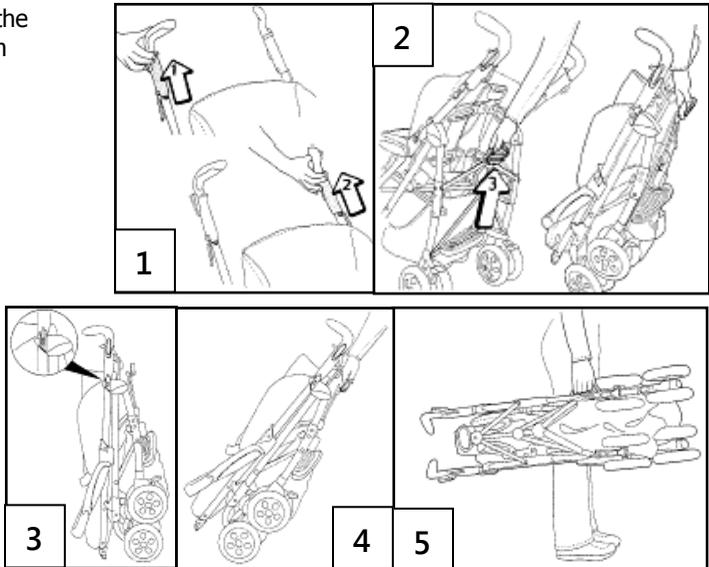

ADJUSTMENTS

Adjustment of handles

The handles are individually adjustable at two different heights; press the block button and raise or lower the handle..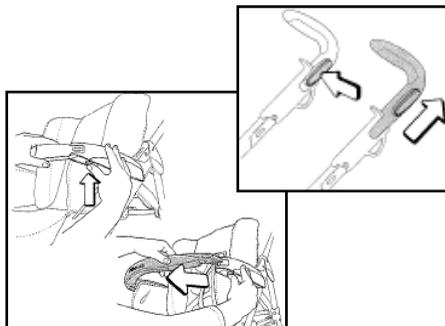

Front bar

Fit or remove the front-bar as shown.


Brakes and castors

The rear wheels have a parking brake system.

It is also possible to block the pirouetting movement of the front wheels using the block system.

Hood assembly

Insert the hood into the side supports and push downwards until it "clicks", as shown; button the hood to the buggy frame and push down on the compass hinge to tighten it.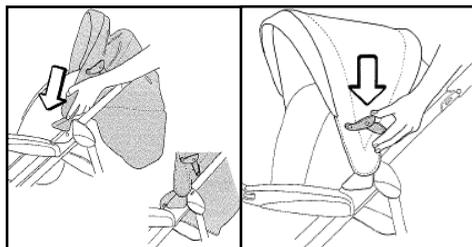


Backrest inclination

The backrest can be adjusted to four different positions.

To lower it, pull up the handle B and at the same time lower the backrest to the required position and block it.


To raise, push the backrest upwards.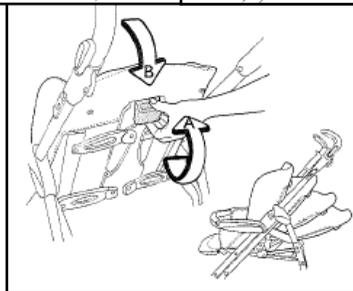

Seat depth and width adjustment

The seat depth can be adjusted by loosening the two bolts positioned on the side runners under the seat and adjusting the seat by following the sliding mechanism. The lateral cushions have a zip in order that extra pads can be inserted so that the seat-width can be narrowed

Warning: if extra pads have been added, when folding the lateral cushions must be moved to the outside of the Pliko buggy.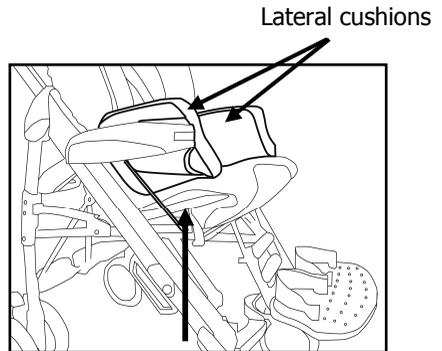

Blocking bolt position of the seat sliding mechanism

Headrest and pelvis supports

The headrest is adjustable at different heights through the holes in the backrest covering.

The pelvis support can be removed if not necessary or be fitted at the required height using the slots in the covering

Foot platform adjustment

The foot platform is height adjustable by loosening the locking bolts and the tilting adjustment is made by screwing or unscrewing the bolt under the platform itself. The platform has two width adjustable footrests.

Belts

The belts with sliding pads can be fixed at the appropriate height simply by inserting them into the right slot of the backrest covering.

To fix the pelvic belt, use the fixing slots on the back of the seat.

Pads and postural systems

The seat, backrest, lateral cushions and pelvic support are made so that it is possible to add pads as necessary to suit the child's postural needs.


All covers are removable and washable, as per the label instructions.

The seat cushion has a Velcro closing system that allows for the raising of the front part if required.

All postural adjustments are to be carried out by specialised personnel, after the evaluation of each single child.

Storage basket

The storage basket must be fixed using the proper hooks (AB-C) as shown in the picture.

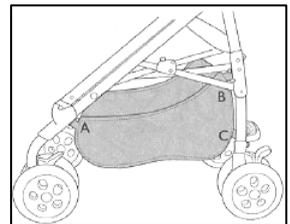fumagalli
WE TAKE CARE

Fumagalli srl
Piazza Puecher 2
22037 Ponte Lambro (Co)
Tel. (+39) 031 33 56 811
Fax (+39) 031 622 111
www.fumagalli.org